



FRx Trainer Instructions

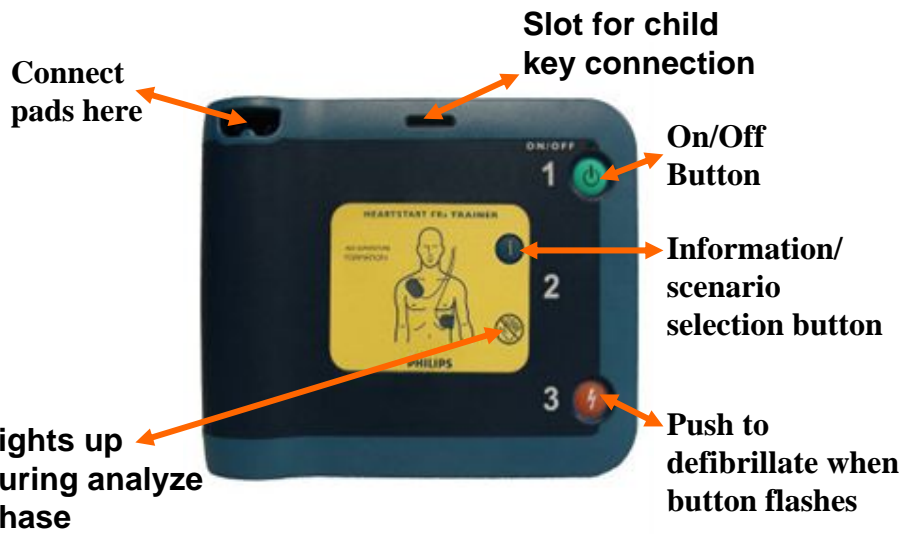

How to use metallic strips if you are using a manikin without permanent metal pins for the apex and sternum

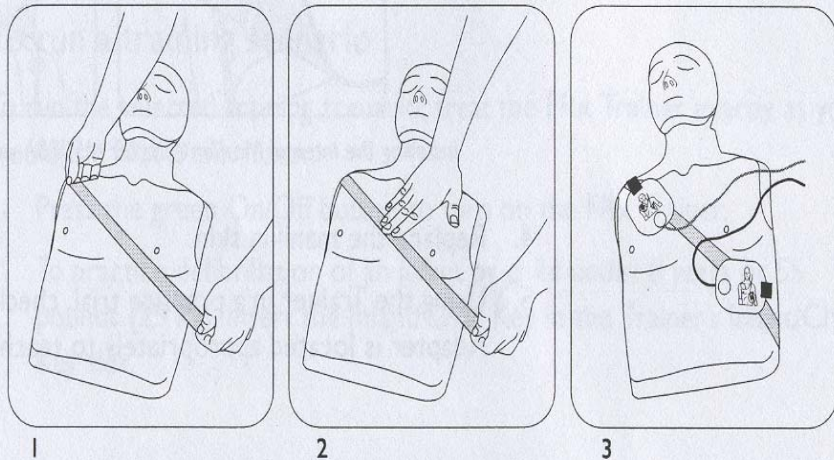

Applying the External Manikin Adapter M5089A to an adult manikin.

Once class is completed remove metallic strip and place back onto the slick side of paper holder

Key Points to remember

- Make sure pad connector is inserted into FRx trainer before turning on
- Push green button to turn on the device and have the student follow the prompts for completion of the AED section of BLS competency. **NOTE**: there is no remote control for the FRx trainer
- Once the student has completed the BLS training push and hold the green on/off button for 2 seconds or the light turns off

Key Points to remember

- Always leave pads connector plugged into FRx trainer
 - a) If pad connector is/ or becomes unplugged and trainer is turned on you will get a message to pick a scenario by pushing the blue flashing button
 - b) To set scenario: push blue flashing button once to pick scenario #one and then push and hold the green on/off button to save scenario #one
- To practice the child scenario, (1-8 years or < 25kg) insert the child key into slot, (diagram attached)
- When scenario is finished push and hold the green on/off button to turn off the trainer
- When closing pad case be sure to place wires back into slots so the wires do not become damaged